



TV Connect  
User Manual

Version 1.0

# Table of Contents

1. Introduction
2. Product Specifications
3. Getting Started
  - Unboxing
  - Device Overview
4. Setup
  - Downloading the TV Connect App
  - Connecting TV Connect to Your Devices
  - Placement Recommendations

5. Using TV Connect with Maggie iVR
6. Troubleshooting
7. Technical Specifications
8. Safety and Maintenance
9. Warranty & Support

# 1. Introduction

The **Maggie iVR TV Connect** is a wireless transmitter that enables users to stream video content from a TV or other HDMI sources directly to their **Maggie iVR** headset. Designed for individuals with low vision, **TV Connect** enhances the viewing experience by magnifying and optimizing images for clarity and comfort. With **TV Connect**, you can enjoy TV shows and movies without the need to sit close to a screen.

## 2. Product Specifications

- **Wireless Connectivity:** Secure transmission to Maggie iVR
- **Compatibility:** Works with streaming devices, cable boxes, Blu-ray/DVD players, computers, and gaming consoles (HDMI output required)
- **Input Ports:** HDMI
- **Output Ports:** HDMI (for optional pass-through to TV)
- **Power Supply:** USB-C

- Range: Up to 50 feet (line of sight)

## **3. Getting Started**

### **Unboxing**

Inside the Maggie iVR TV Connect box, you will find:

- **TV Connect Transmitter**
- **Power Adapter (USB-C)**
- **1 HDMI Cable**
- **User Manual**

# Device Overview

- **HDMI Input Port:** Connects to your video source (streaming device, cable box, etc.)
- **HDMI Output Port:** Optional pass-through to a TV or monitor
- **USB-C Power Port:** Connects to the included power adapter



## 4. Setup

### Step 1: Download the TV Connect App

Before setting up TV Connect, download the “**TV Connect**” app from the Apple App Store.

**Note:** You do **not** need to open this app—it just needs to be installed on your iPhone for TV Connect to function properly.

## **Step 2: Connecting TV Connect to Your Devices**

### **1. Remove the Streaming Device from Your TV**

- Locate the HDMI cable connecting your streaming device (Roku, Apple TV, Fire Stick, etc.) to your TV.
- Unplug this HDMI cable from the streaming device but leave it connected to your TV.

## **2. Connect HDMI to the TV Connect Box**

- Take the HDMI cable you just removed from the streaming device and plug it into the HDMI OUT port on the TV Connect box.

## **3. Connect the TV Connect Box to Your Streaming Device**


- Using the provided HDMI cable, plug one end into the HDMI IN port on the TV Connect box.

- Plug the other end into the HDMI port on your streaming device.

#### **4. Power the TV Connect Box**

- Plug the USB-C cable into the TV Connect box.
- Connect the other end of the USB-C cable into a wall power outlet using the provided adapter.

The following image is a diagram for assistance.

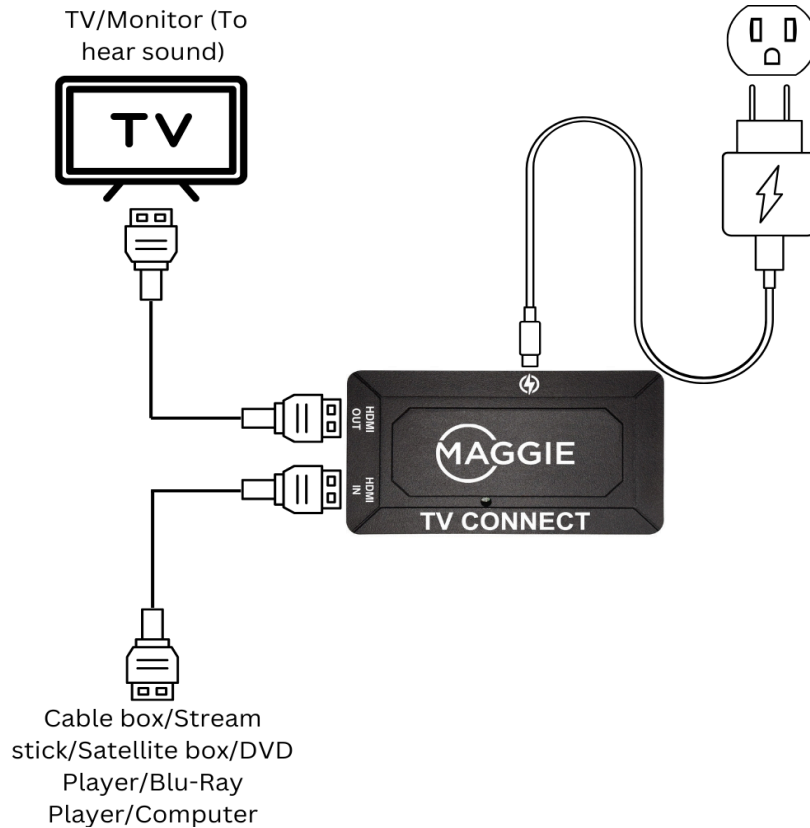

## Step 3: Turn on the TV and Set Up Maggie iVR

1. Turn on your TV.
2. Put on your Maggie iVR headset.
3. Cycle through the Modes by pressing the M button on the headset until you see the TV Connect icon.



4. Select the TV Connect Mode by pressing either the + or - button.
5. Your TV should now stream directly to your headset.

## 5. Using TV Connect with Maggie iVR

Once TV Connect is successfully set up:

- Use the **zoom buttons (+/-)** on your Maggie iVR to adjust the screen size.
- Enjoy hands-free, comfortable TV viewing from any seating position.
- To reset the view of the screen within the headset, hold down the M button on the headset until the view resets.



## 6. Troubleshooting

**Issue:** No signal detected on Maggie iVR

- Ensure TV Connect is powered on and properly connected.
- Check that your video source (streaming device, cable box, etc.) is turned on and outputting to the TV.
- Ensure Maggie iVR is in TV Connect mode in the app.


## **Issue:** Weak or distorted signal

- Move TV Connect to a higher or more open location.
- Ensure there are no major obstructions between the transmitter and the Maggie iVR headset.
- Ensure you are within 40-50 feet of TV Connect (line of sight).

## 7. Technical Specifications

- **Dimensions:** [Little bitty]
- **Weight:** [Not much]
- **Power Input:** USB-C, 5V/3A
- **Operating Range:** Up to 50 feet
- **Operating Temperature:** 32° to 104°F (0° to 40°C)

## 8. Safety and Maintenance

 **Warning:** To ensure optimal performance and longevity of TV Connect, follow these safety guidelines:

- Place TV Connect on a flat, clean, dry surface and ensure it is well ventilated.
- Minimize dust buildup by keeping the area around TV Connect clear.

- Do not block ventilation openings—this can cause overheating.
- Use only the provided USB-C power adapter to avoid damage.
- Clean with a soft, dry cloth—avoid harsh chemicals.

## 9. Warranty & Support

Maggie's TV Connect comes with a one-year limited warranty. For support, visit our website at [www.maggie.vision](http://www.maggie.vision) or contact our customer service team at [support@maggie.vision](mailto:support@maggie.vision).